

Order Information

ENDORAIL SYSTEM

ORDER NUMBER	PACKAGING	DESCRIPTION
ESYSTEMGEN2V01	box of 1 unit reusable	Endorail Handpiece and Endorail Cart

ENDORAIL SET

ORDER NUMBER	PACKAGING	DESCRIPTION
ESETGEN2V02	box of 4 units single use	Endorail Balloon Guide Endorail Solution Syringe

Compatible with endoscope tool channel diameter equal or higher than 3,2 mm.

Please consult product labels and package inserts for indications, contraindications, hazardous, warnings, cautions and instructions for use.

The Product is CE marked.

Manufactured by
Endostart s.r.l.

Via delle Regioni, 265 50052 Certaldo (FI) Italy
+39 0571 843033 | C.F. 06835050482
info@endostart.com | www.endostart.com



Endostart s.r.l. © All Rights Reserved

ENDO
RAIL

THE GUIDED WAY TO COLONOSCOPY



Magnetic Balloon Technology

The solution for challenging looping


No setup costs

Compatible with standard colonoscopes

On demand usage and no need of preloading



Magnetic Balloon Technology

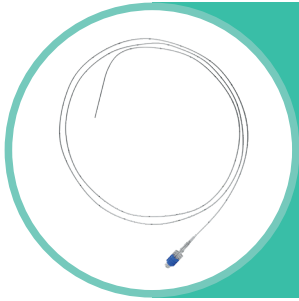

ENDORAIL BALLOON GUIDE IS A SINGLE USE BALLOON CATHETER.

It can be inserted in the colonoscope tool channel and filled with Endorail Ferromagnetic Fluid.


ENDORAIL FERROMAGNETIC FLUID IS A WATER DISPERSION OF BIOCOMPATIBLE IRON POWDER.

It can be easily injected inside the catheter for filling the balloon.



ENDORAIL HANDPIECE CONTAINS A PERMANENT MAGNET.

Its design allows to easily modulate the intensity of the magnetic field.

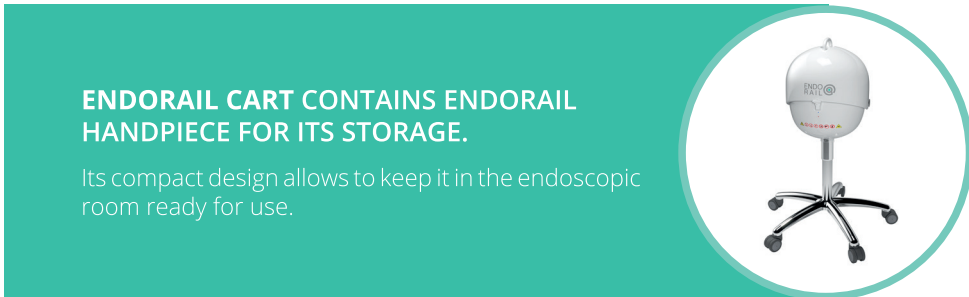

ENDORAIL CART CONTAINS ENDORAIL HANDPIECE FOR ITS STORAGE.

Its compact design allows to keep it in the endoscopic room ready for use.

Easy and quick procedure!

Sometimes solving loops is challenging.



Endorail Balloon Guide is inserted in the colonoscope tool channel. The balloon is filled with Endorail Ferromagnetic Fluid.

Endorail Handpiece is placed over the patient's abdomen. The balloon is magnetically anchored.



Then edoscopist pulls and rotates the colonoscope to solve the loop.

The balloon is deflated, removed and colonoscopy is completed with standard technique.

